

Lubricating Jelly

Instruction

1. Product Name: Lubricating Jelly
2. Component: Purified Water, Glycerin, Kaposi and other medical materials.
3. Intended Use
The medical devices before checking the intestinal tract, intubation and vaginal were smeared by the lubricating jelly to increase the lubricity and reduce the irritation.
4. Specification: Lub0005, Lub0010, Lub0020
5. Symbols
The meaning(s) of the symbols shown on the package, the back cover of this instruction manual and/or this instruction are as follows:



DO NOT REUSE



BATCH CODE




DATE OF MANUFACTURE



USE BY



MANUFACTURER

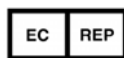

AUTHORISED REPRESENTATIVE IN THE EUROPEAN COMMUNITY



KEEP AWAY FROM SUNLIGHT



KEEP DRY



DO NOT USE IF PACKAGE IS DAMAGED



NOTIFIED BODY

Wilson Instruments (SHA) Co., Ltd.

6. Operation

Rive off the easy tearing notches or aluminum membrane, and squeeze out proper lubricating jelly to smear the place you want. Because the product is a water-based lubricant, it is easy to clean up after use.

7. The life time

Deadline of the products is for three years. The seal shows the batch code.

8. Caution


- 1) The lubricating jelly is only for single use.
- 2) Prohibit the use of lubricating jelly if the small packing is destroyed.
- 3) Repeated use will cause cross infection


9. Storage

The products should be store in the shadow at room temperature and dry place, avoiding direct sunlight. The limit temperature range of the product is $-5^{\circ}\text{C}\sim 60^{\circ}\text{C}$.

10. Handling

The lubricating jelly is disposable. Do not reuse. Repeated use will cause the risk of infection. Follow the regulatory requirements to dispose of the packing used properly to ensure safe.

 **Manufacturer: Wilson Instruments (SHA) Co., Ltd.**
Address: 25D, He Yi Business Plaza No.420, Jiang Ning Road, Shanghai,
P. R. China
Zip Code: 200041
Phone: 0086-21-66311471
Fax: 0086-21-66311472
Website: www.heyinovo.com

 **European Representative: Lotus Global Co., Ltd.**
Address: 15 Alexandra Road, London, NW8 0DP, United Kingdom
Phone: +0044-20-75868010
Fax: +0044-20-79006187

File No.: WI-RD-12-012 Rev.01

Add: 25D, He Yi Business Plaza No.420, Jiang Ning Road, Shanghai, P. R. China
TEL: ++86-21-66311471 FAX: ++86-21-66311472